


STAFF APP INSTALLATION INSTRUCTIONS




To **download**, please proceed with the following steps:



- Click on the link <http://cltrainingtool.com/app/>
- OR Select the **Safari** icon and type in the link above
- Tap the install **The Staff to Patient Contact Lens App**

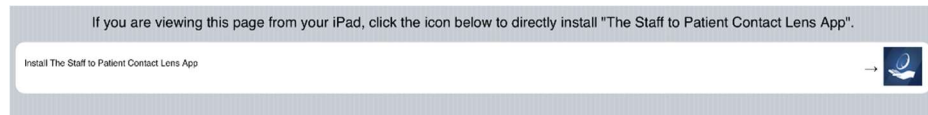


- Select install

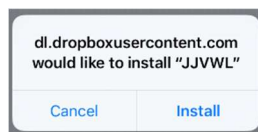

* If you receive a **“Untrusted Enterprise Developer”** error, please proceed with the following steps to ‘trust’ the app.



- Select the Setting icon
- Choose General

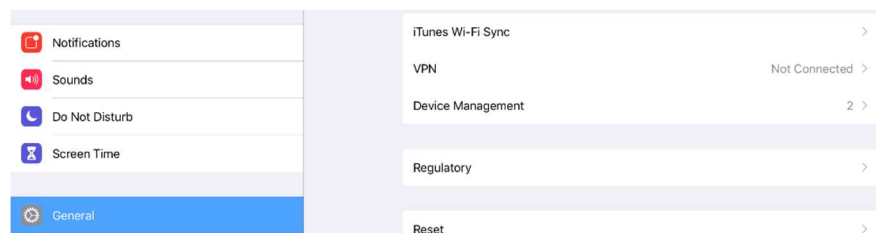

- Scroll down to select Device Management



- Select Red E Technologies, LLC to ‘Trust’ it

If you have any issues, please email Thomas Estes at test321@its.jnj.com